

Application instructions



1. Open outer packaging.



2. Remove shower pouch.



3. Hold pouch with tape to the top of pouch. Insert catheter into the pouch opening.



4. Remove lower strip.



5. Remove upper strip.



6. Stick firmly to the skin around catheter dressing. To help form a good seal shave or clip hair.

(if present and possible to do so)

to site where pouch adhesive will be applied,

ensure area is clean and dry, and free from creams or moisturisers prior to application.



submed Safe Easy

Su Biyomedikal Sistemler ve Sağlık. Hiz. San. ve Tic. A.Ş.
Orhangazi Mah. 1673 Sok. No:20 Esenyurt/ İstanbul/ Turkey
T: +90 212 320 37 53 F:+90 212 320 53 51 info@submed.com.tr
www.submed.com.tr

LINC
Medical

Distributor : LINC Medical Systems Ltd.
LINC Medical Systems Ltd 7 Kingsley Street Leicester LE2 6DY
T: +44 (0) 1572 717515 sales@linc-medical.co.uk
www.linc-medical.co.uk

CE